

Postal Pack, for UN3373 Biological Substances

Conforms with Packing Instruction 650




BioBag 751 (Code 751)

Int Dims
160 x 230mm
Folded Int Dims
155 x 105mm

4-Bay Absorbent Cushioning Pouch (Code 740)

200 x 115mm
absorbs 120ml
of fluid

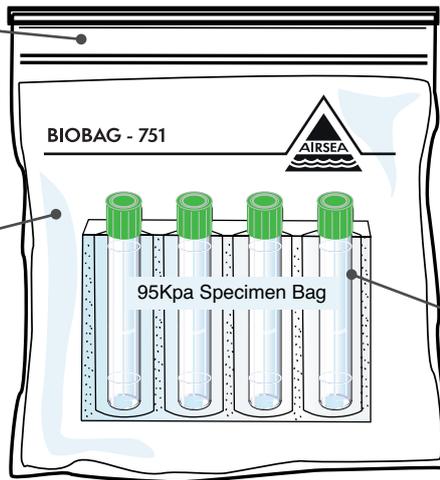

Typical Vials
(Not Supplied)

Postal Pack

Foil Tamper
Evident Seal
(Code 741)



ASSEMBLY DETAILS:

Postal Pack Code: 760

Box Dimensions:

External 210 x 125 x 30mm
Internal 190 x 115 x 26mm

Instructions:

- Place the primary receptacle(s) into the cushioning pouch.
- Place into the 95kPa bag and seal as per printed instructions.
- Fold and place sealed bag into box.
- Seal with tamper evident seal.

Notes:

- The Postal Pack is designed to be delivered via regular Royal Mail.
- The Postal Pack comes complete with fibreboard outer, 4-Cell Absorbent Pouch, 95Kpa Biobag 751, & tamper evident foil security seal.

Thermal control:

A variety of suitable Thermal Control Units are available.

It is the Shippers responsibility to ensure that packagings conform to the relevant transport regulations.

Further info



CAUTION

This packaging is NOT a U.N. Certificated Packaging for the transport of class 6.2 Infectious Substances, Category A and does not conform to Packing Instruction 620.

Air Sea Containers Ltd

521 Cavendish Street, Merseyside, CH41 8FZ, UK

▲ Tel +44 (0)151 653 1500 ▲ Email uk@airseadg.com ▲ Visit www.airseadg.com