


BioShipper-1, for UN3373 Biological Substances

Conforms with Packing Instruction 650



BioBag 751

160 x 230mm
Int Dims
(Code 751)
1 bag supplied,
additional bags
sold separately.

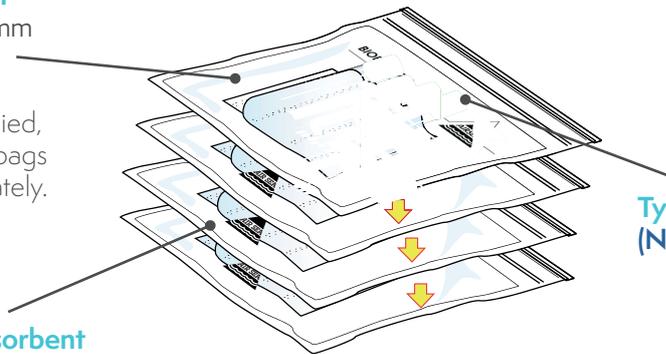

Typical Vials
(Not Supplied)

4-Bay Absorbent Cushioning Pouch


200 x 115mm
(Code 740)
absorbs 120ml
of fluid



Sufficient absorbent material
must be used to absorb the
entire contents and cushioning
to prevent breakage.

BioShipper-1

243 x 180 x 85mm
Int Dims
(Code 761)

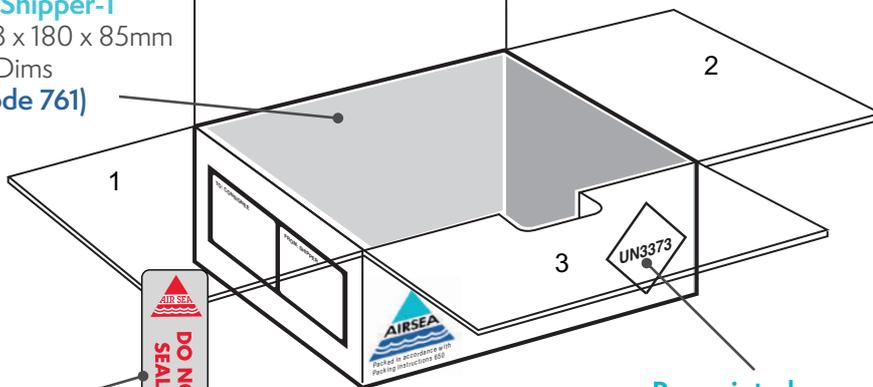

Pre-printed
UN3373 Label

Foil Tamper
Evident Seal
(Code 742)



Add additional cushioning to fill void spaces

ASSEMBLY DETAILS:

BIOSHIPPER-1 Code: 761

Box Dimensions:

External 250 x 190 x 95mm
Internal 243 x 180 x 85mm

Instructions:

- Place the primary receptacle(s) into the cushioning pouch(s).
- Place into the 95kPa bag and seal as per printed instructions.
- Place sealed bag(s) into BioShipper-1.
- Seal with tamper evident seal.

Notes:

The BioShipper-1 is supplied flat packed, it's designed with an easy pop up construction and is capable of carrying up to 4x BioBag 751.

The BioShipper-1 comes complete with:
Code 751, 95Kpa BioBag 751
Code 742, Security Seal
Code 740, 4 Bay Absorbent
Outer Box with Pre-Printed
UN3373 Label

Thermal control:

A variety of suitable Thermal Control Units are available.

It is the Shippers responsibility to ensure that packagings conform to the relevant transport regulations.

Further info



CAUTION

This packaging is NOT a U.N. Certificated Packaging for the transport of class 6.2 Infectious Substances, Category A and does not conform to Packing Instruction 620.

Air Sea Containers Ltd

521 Cavendish Street, Merseyside, CH41 8FZ, UK

▲ Tel +44 (0)151 653 1500 ▲ Email uk@airseadg.com ▲ Visit www.airseadg.com