


BioShipper-2, for UN3373 Biological Substances

Conforms with Packing Instruction 650




BioBag 752
230 x 315mm
Int Dims
(Code 752)

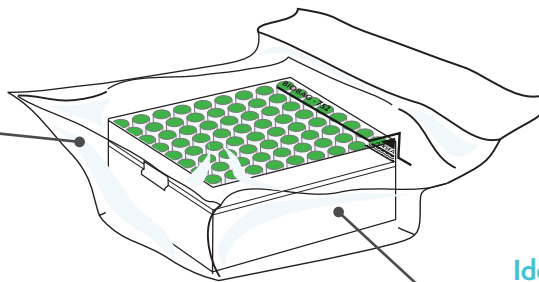

**Ideal for cryovial
boxes or a
multitude
of single primary
containers
(Not Supplied)**

BioShipper-2
315 x 273 x 101mm
Int Dims
(Code 762)

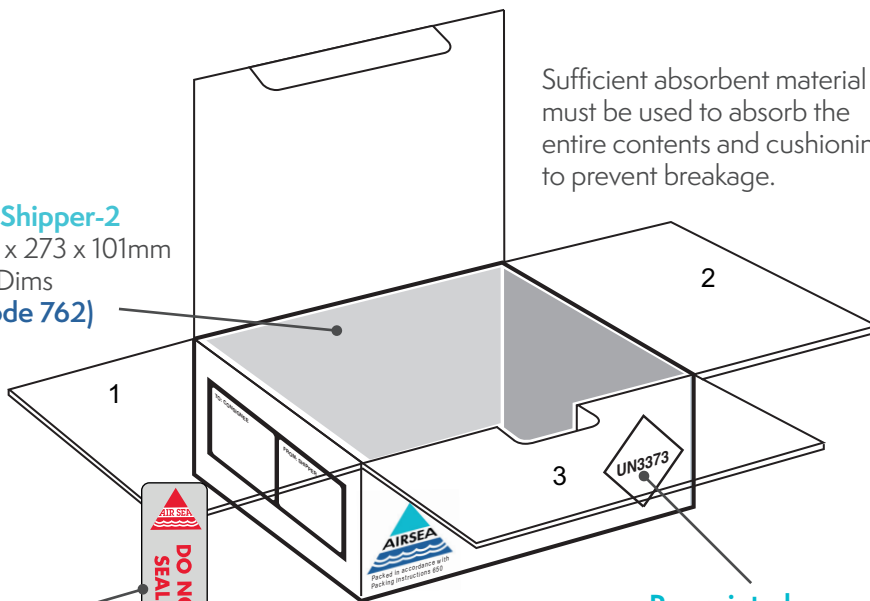

**Foil Tamper
Evident Seal
(Code 743)**

**Pre-printed
UN3373 Label**

Add additional cushioning to fill void spaces

Sufficient absorbent material
must be used to absorb the
entire contents and cushioning
to prevent breakage.

ASSEMBLY DETAILS:

BioShipper-2 Code: 762

Box Dimensions:

External 318 x 276 x 110mm
Internal 315 x 273 x 101mm

Instructions:

Place primary receptacle(s) into the cushioning pouch(s). If using cryovial use absorbent sheet

Place into the 95kPa bag and seal as per printed instructions.

Place sealed bag(s) into BioShipper-2. Seal with tamper evident seal.

Notes:

The BioShipper-2 is capable of carrying numerous biobag-752 containing cryovial boxes or multiple single primary receptacles.

The BioShipper-2 comes complete with:

Code 752, 95Kpa BioBag
Code 743, Security Seal
Code 740, 4 Bay Absorbent
Cushioning Pouch x 2 (240ml)
Outer Box with Pre-Printed
UN3373 Label

Thermal control:

A variety of suitable Thermal Control Units are available.

It is the Shippers responsibility to ensure that packagings conform to the relevant transport regulations.

Further info



CAUTION

This packaging is NOT a U.N. Certificated Packaging for the transport of class 6.2 Infectious Substances, Category A and does not conform to Packing Instruction 620.

Air Sea Containers Ltd

521 Cavendish Street, Merseyside, CH41 8FZ, UK

▲ Tel +44 (0)151 653 1500 ▲ Email uk@airseadg.com ▲ Visit www.airseadg.com