

Postal Pack, for UN3373 Biological Substances

Conforms with Packaging instruction 650




BioBag 751 (Code 751)

Int Dims
160 x 230mm
Folded Int Dims
155 x 105mm

4-Bay Absorbent Cushioning Pouch (Code 740)

200 x 115mm
absorbs 120ml
of fluid

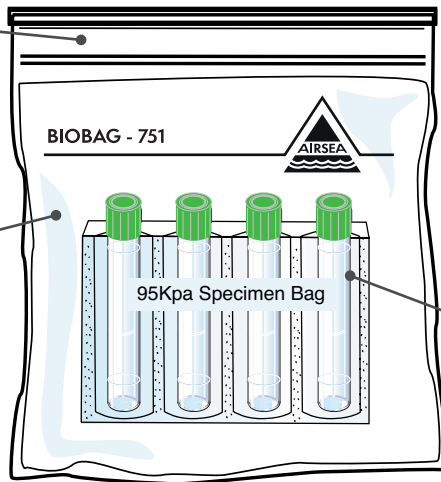

Typical Vials
(Not Supplied)

Postal Pack

Foil Tamper Evident Seal (Code 741)



ASSEMBLY DETAILS:

Postal Pack Code: 760

Box Dimensions:

External 210 x 125 x 30mm
Internal 190 x 115 x 26mm

Instructions:

Place the primary receptacle(s) into the cushioning pouch.

Place into the 95kPa bag and seal as per printed instructions.

Fold and place sealed bag into box.

Seal with tamper evident seal.

Notes:

The Postal Pack is designed to be delivered via regular Royal Mail.

The Postal Pack comes complete with fibreboard outer, 4-Cell Absorbent Pouch, 95Kpa Biobag 751, & tamper evident foil security seal.

Thermal control:

A variety of Thermal Control Units for transportation in dry ice or coolpack are available.

It is the Shippers responsibility to ensure that packagings conform to the relevant transport regulations.

Further info



CAUTION

This packaging is NOT a U.N. Certificated Packaging for the transport of class 6.2 Infectious Substances, Category A and does not conform to Packaging Instructions 620.

Air Sea Containers Ltd

521 Cavendish Street, Merseyside, CH41 8FZ, UK

▲ Tel +44 (0)151 653 1500 ▲ Email uk@airseadg.com ▲ Visit www.airseadg.com